


INSTALLATION

OF THE KIT E-JOELETTE NEW GENERATION

CONTENTS OF THE KIT

www.joeletteandco.com

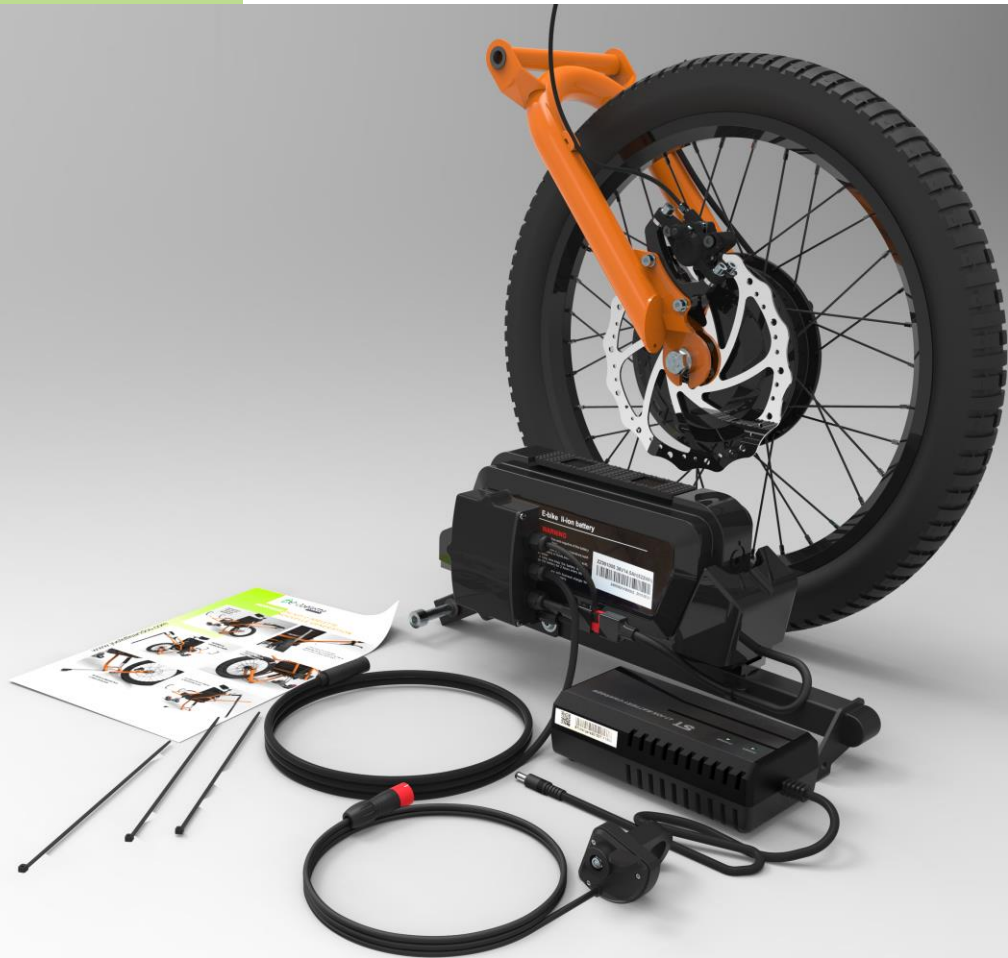

ASSEMBLY STEPS

PUT THE
JOELETTE IN
« TRANSFER
MODE »



REMOVE ALL OF THE CLIPS THAT
IMMOBILIZE THE BREAK HOSE
(EXCEPT ON THE SWINGARM)



REMOVE THE
BREAK LEVER



DISCONNECT THE
SHOCK ABSORBER IN
THE LOWER PART



REMOVE THE SPINDLE AND
THE SWINGARM




THE JOELETTE IS READY
TO RECEIVE THE KIT





CONNECT THE KIT TO THE TRANSFER FRAME WITH THE SPINDLE

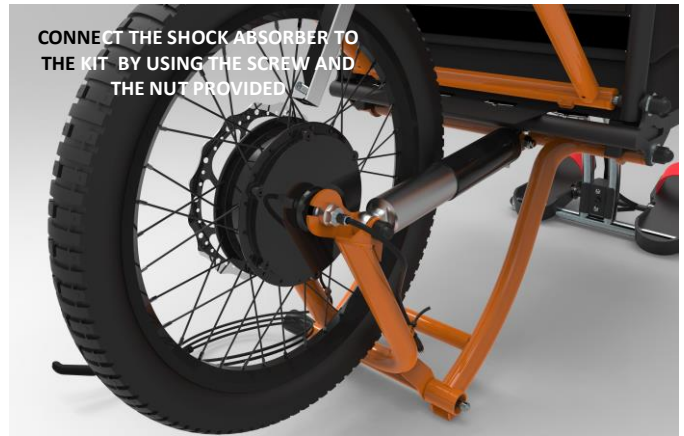

CONNECT THE SHOCK ABSORBER TO THE KIT BY USING THE SCREW AND THE NUT PROVIDED




MOUNT THE ACCELERATOR AND THE BRAKE LEVER PROVIDED AND REPLACE THE CLIPS IN ORDER TO IMMOBILIZE THE HOSE AND THE ELECTRIC CABLE




TAKE THE BATTERY OFF ITS SUPPORT



IN THE HIGHER PART, MOUNT THE SUPPORT ON THE JOELETTE WITH THE M10X80 SCREW PROVIDED ...

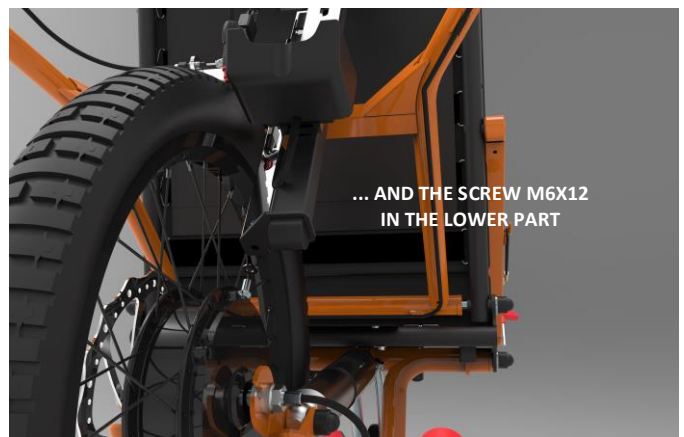


... AND THE SCREW M6X12 IN THE LOWER PART

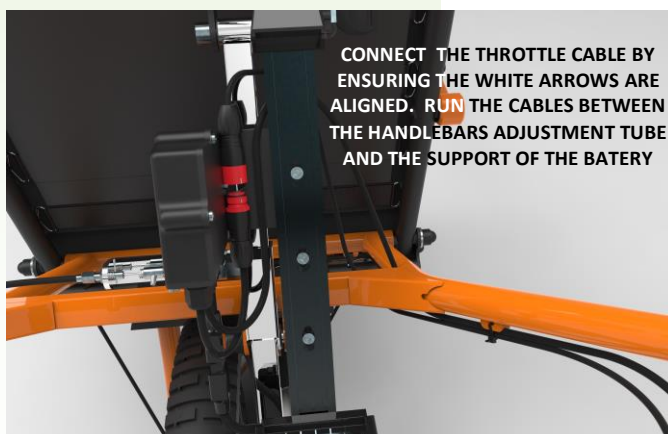

CONNECT THE THROTTLE CABLE BY ENSURING THE WHITE ARROWS ARE ALIGNED. RUN THE CABLES BETWEEN THE HANDLEBARS ADJUSTMENT TUBE AND THE SUPPORT OF THE BATTERY



CONNECT THE MOTOR POWER SUPPLY CABLES BY ALIGNING PROPERLY THE ARROWS

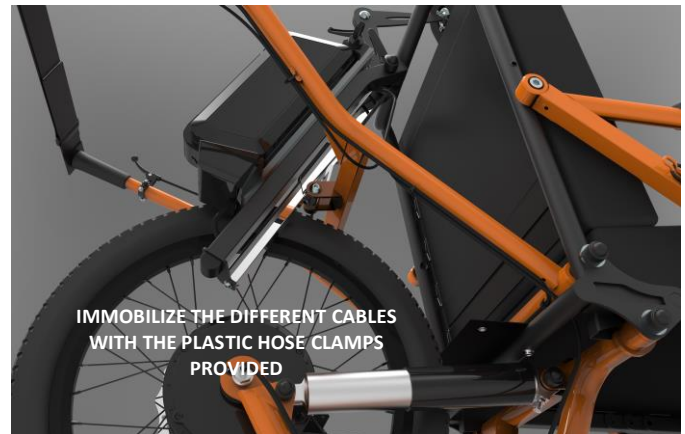

Specific information

- For the maintenance and precautions for use, read the general manual attached to the package of the **one-wheel Joëlette**.
- To make the transfer of disabled persons from their wheelchair to the Joëlette TWIN, apply the same procedure as for **the classic one-wheel model**.
- Before going on a hike, make sure each safety pin is in place.

