

INSTALLATION OF THE KIT JOELETTE TWIN NEW GENERATION

PUT THE
JOELETTE IN
TRANSFER
MODE



REMOVE ALL THE CLIPS THAT
IMMOBILIZE THE BRAKE HOSE
(EXCEPT ON THE SWINGARM)




REMOVE THE
BRAKE LEVER



DISCONNECT
THE SHOCK ABSORBER
IN THE LOWER PART



REMOVE THE SPINDLE AND
THE SWINGARM

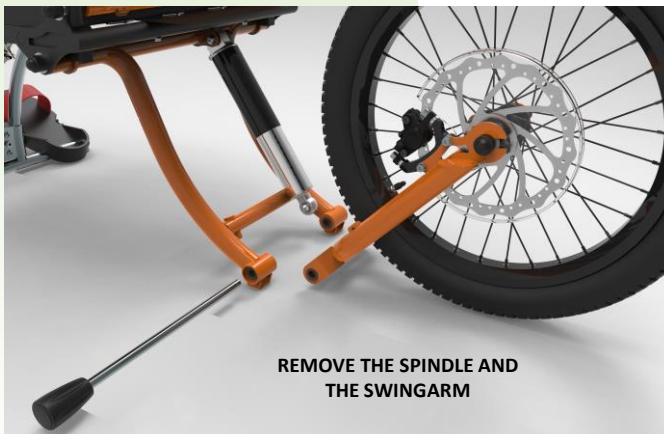


THE JOELETTE IS READY
TO RECEIVE THE KIT





CONNECT THE KIT TO THE TRANSFER FRAME WITH THE SPINDLE

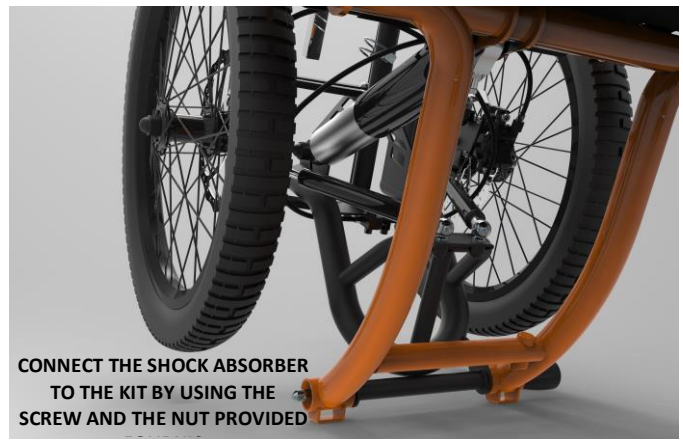

CONNECT THE SHOCK ABSORBER TO THE KIT BY USING THE SCREW AND THE NUT PROVIDED



MOUNT THE BRAKE LEVER PROVIDED AND REPLACE THE CLIPS IN ORDER TO IMMOBILIZE THE HOSE

Specific information:

- For the maintenance and precautions for use, read the general manual attached to the package of the **one-wheel Joëlette**.
- To make the transfer of disabled persons from their wheelchair to the Joëlette TWIN, apply the same procedure as for **the classic one-wheel model**.
- If necessary you can remove the Jockey wheel by turning it 180°C.
- Before going on a hike, make sure each safety pin is in place.
- During the first use, take the time to evaluate the itinerary and the crossing possibilities of the **Joëlette TWIN**.



YOUR  IS READY